

TAKTVOLL®


VUE 3 SUPER ENERGY PLATFORM

Simplify Complexity with High Integration



Taktvoll proudly presents the next-generation Super Electrosurgical Workstation – the VUE 3 Energy Platform. With a highly integrated and innovative design, it combines multiple functions, including large vessel sealing, ultrasonic scalpel, bipolar electrosurgery, and monopolar electrosurgery, into one device. This revolutionary platform provides healthcare professionals with a more flexible surgical experience, a more comfortable operating environment, a more efficient management model, and more cost-effective operational solutions.

SUPER ENERGY PLATFORM

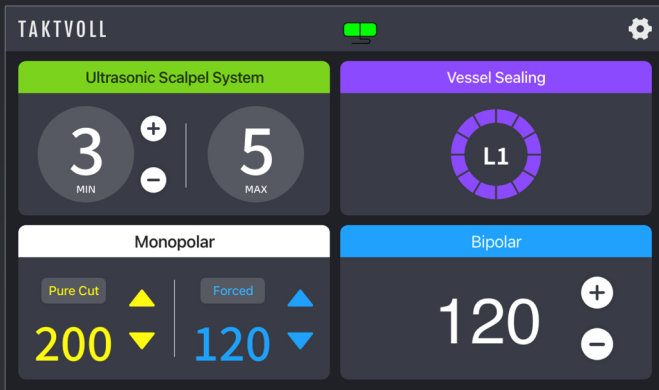

- Glass full version design, which is extremely tense and beautiful
- Color touch screen operation panel, flexible and easy to operate
- Highly Integrated · All-in-One Solution
- High Performance Ultrasonic Scalpel System
- Large blood vessel sealing function
- Electrosurgical Pens and Foot Switch Control
- Recently Mode, Power and Other Parameters
- Quick Adjustment of Power
- Cut and Coag in an Intermittent Manner



3 Cutting Modes

PURE Provides clean and precise tissue cutting.

BLEND 1 Provides a combination of 70% Cutting and 30% Coagulation

BLEND 2 Provides a combination of 50% Cutting and 50% Coagulation

2 Coagulation Modes

SPRAY Designed to provide hemostasis in a broader area with less depth of tissue damage when compared to desiccation.

FORCED Fast and effective standard coagulation with moderate thermal penetration. Slight carbonization effects.

Vessel Sealing Mode

Vessel Sealing

Seal and cut vessels up to and including 7mm in diameter. Sealing vessels and tissue bundles without dissection or isolation, easy to operate.

Bipolar Mode

Standard

Bipolar coagulation which gently coagulates tissue for minimal tissue damage reducing carbonization and sticking.

Ultrasonic Scalpel Mode

Next-generation adaptive tissue technology, effectively addressing various challenges in open and laparoscopic surgeries. Capable of sealing blood vessels with a maximum diameter of 5mm.

TAKTVOLL[®]
VUE 3

Beijing Taktvoll Technology Co., Ltd.

Address: Room 302, 3rd Floor, Building 10, No. 13, Jingsheng South 4th Street, Beijing Economic and Technological Development Zone, Beijing, China.

Tel: +86(10) 56495579 / 56495580 Web: www.taktvollmed.com E-mail: Support@taktvoll.com.cn

